BITE FORK SET

BISSGABEL SET

DE GEBRAUCHSINFORMATION

EN INSTRUCTIONS FOR USE

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BITE FORK SET

The bite fork set is a reliable instrument for checking the correct position of the occlusal plane (Fig. 1). The various planes can be clearly defined with the vertical pins and the rubber band. One of the most important parameters in prosthetics is the parallelism between the occlusal plane and Camper's plane or the bipupillary line (Fig. 2–3). Additional reference planes such as the Frankfurt horizontal plane and the axial-orbital plane can also be checked.

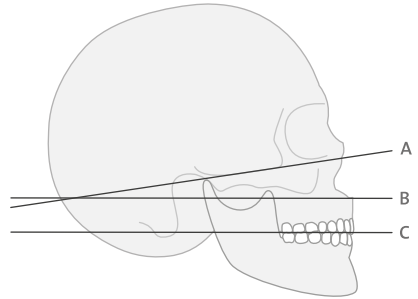
Fig. 1 Various planes on the cranium bone

A Axial-orbital plane

B Camper plane

C Occlusal plane

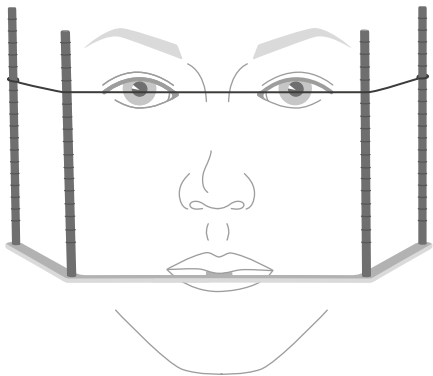
Fig. 2 Shows the bipupilar line

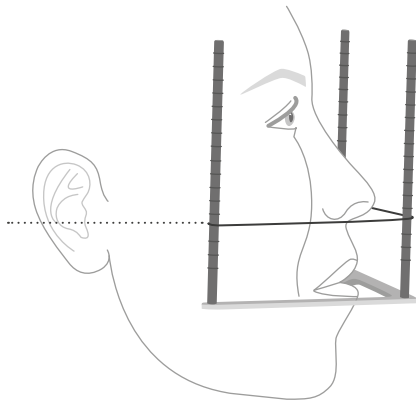
Fig. 3 Shows the Camper plane

Indication

Checking of the correct position of the occlusal plane for registrations and esthetic templates for partial and total prosthetic restorations.

Contraindication

The bite fork cannot be used for restorations of fully dentulous jaws.

Application

- > Screw the 4 vertical pins into the threads of the bite fork provided for this purpose.
- > Dimension the rubber band so that it can be loosely tensioned over all 4 vertical pins at the desired height.
- > Insert the bite registration in the patient's mouth, then align it together with the bite fork on the maxillary wax wall. This allows the wax walls of the maxilla registrations to be checked for their correct alignment to the occlusal plane.
- > Corrections to the wax wall can be made with the CANDULOR Rim Former.

Notes on cleaning

What	When	With what
Bite fork	After each use	Sterilization in the autoclave at 134° C for 5:30 minutes
Vertical pins	After each use	Sterilization in the autoclave at 134° C for 5:30 minutes
Rubber band	After each use	Disposal

Warning

The bite fork set was developed for use in the dental sector. Initial operation and handling must be conducted according to the instructions for use. CANDULOR does not accept liability for damages caused by use other than directed or improper handling.

The user is also responsible for confirming that the bite fork set is suitable for the intended purpose before use, particularly if the purpose is not listed in the instructions for use.

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🏢 **CANDULOR AG**