**COLD**

# C-PLAST

POLYMER/MONOMER

**INSTRUCTIONS FOR USE**

---

GEBRAUCHSINFORMATION

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---

Dear customer,

The product that you have purchased qualifies as a medical device under European Directive 93/42/EEC. Please ensure that you file the lot number and product name in your administrative system on receipt of the product. For each piece of work that you produce, please make a note of all the materials used along with all the appropriate lot numbers. For your own protection as well as for the protection of your patients and the environment, please also follow the important guidelines in the following instructions for use.

### **Product description**

C-PLAST cold-curing tray material is ideal for the production of custom trays, base plates and registration plates.

In conjunction with the corresponding adhesive, C-PLAST can be used with most impression materials.

To obtain the best results, please follow the instructions contained in this leaflet.

### **White**

C-PLAST is a cold-curing precision tray material for the production of registration plates, custom trays and precision base plates for bite rims.

### **Color 34**

C-PLAST is a cold-curing precision tray material which is particularly well suited to the production of base plates for trying in full dentures.

### **Composition / Powder**

Polymethyl methacrylate, polyethyl methacrylate, copolymer, calcium carbonate, catalyst, pigments

### **Composition / Liquid**

Methyl methacrylate, paraffin oil, catalyst

## Storage

- Permitted storage temperature: 2–28 °C / 36–82 °F
- Protect from light
- Do not use products after the stated expiry date.
- Keep out of the reach of children.
- Store the monomer container in a dark, cool and well-ventilated place.

## Indication

- Custom trays
- Custom base plates for: bite rims, intraoral registration plates, setting up teeth with wax
- Production of custom incisal guiding plates

## Contraindication

- Do not apply C-PLAST in the oral cavity
- Not suitable for alginates and hydrocolloids
- In the event of an allergic reaction to C-PLAST ingredients, stop using the product immediately.

## Side effects

No systemic side effects have been reported to date.

### **Danger warnings**

- Monomer contains methyl methacrylate (MMA)
- MMA is an irritant and easily flammable (flash point: + 10 °C / 50 °F)
- MMA and its vapours are irritating to the eyes, skin and respiratory system
- May cause sensitisation upon skin contact
- Do not inhale vapours
- Keep away from sources of ignition – no smoking

- Do not empty into drains
- Avoid contact of the skin with monomer and uncured material. Many commercial gloves, e. g. those made of latex or vinyl, are not monomer-resistant and therefore do not provide effective protection against the sensitising effect of methacrylates.
- Wear a mask when grinding and use a suction removal system.
- Always use a spatula when handling the mixture.
- Take precautionary measures against static discharges.

## Important

This material has been developed solely for use in dentistry and must be handled strictly in accordance with the instructions for use. The manufacturer cannot be held liable for any damages arising as a result of failure to observe the instructions for use or the stated area of application. The same applies in the event that the product is mixed or processed with other manufacturers' products. The user shall be solely responsible for testing the material with respect to its suitability prior to use for any purpose other than those explicitly stated in the instructions.

**Safety data sheets can be found on our website at [www.candulor.com](http://www.candulor.com).**

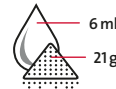
## INSTRUCTIONS FOR USE

### 1. Preparation

The plaster surfaces must be well wetted. Isolate the surfaces with two applications of ISO-K and allow to dry thoroughly.

### 2. Dosage

Mixing ratio for one complete tray:  
21g polymer : 6 ml monomer

Blend the powder and liquid mixture thoroughly in the mixing cup using a spatula.

### 3. Working time

As soon as the material no longer sticks, it can be processed for approx. 5 minutes at 23 °C / 73 °F. The higher the room temperature, the shorter the working time.

The dough can be used directly on the model or placed between two polyethylene sheets and rolled out to form an even layer (ideal thickness 3–4 mm).

Shape the rolled out dough to fit the isolated plaster model, using a pair of scissors or scalpel to remove the excess.

### 4. Polymerisation

The polymerisation time is approx. 7 minutes (at 23 °C / 73 °F) after mixing. Lift the polymerised pieces from the model. Trim the edges.

#### **Repairs and corrections of C-PLAST**

Repairs and corrections may be carried out using C-PLAST. The contact surfaces must be well roughened with a bur and wetted with monomer.

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🏢 Candulor AG