# KMG PASTE

DE GEBRAUCHSINFORMATION

EN INSTRUCTIONS FOR USE

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**Product description**

KMG Paste is used to polish dental resins, composites and metals. The polishing paste can be used for prepolishing and high-gloss polishing in combination with brushes (eg goat hair) and polishing buffs (eg leather).

**Composition**

Aluminium oxide dispersed in an aqueous solvent emulsion.

**Composition – Polishing of**

- > resin or composite restorations
- > resin parts of partial and complete restorations
- > metal restorations (eg inlays, onlays, crowns, etc)

**Restrictions**

- > Polishing of metal-ceramic restorations
- > Application of KMG Paste in the mouth

**RESTRICTIONS****Resin restorations**

Depending on the desired surface texture, the restoration is prepolished with KMG Paste and goat hair brushes. Subsequently, the restoration is polished to a high lustre with KMG Paste using a cotton or leather buffing wheel. Depending on the type of polish desired, leather buffing wheels can be used to achieve a high shine, while cotton buffs are used to achieve a lesser degree of lustre.

**Metal restorations**

Metal is polished using felt wheels or stiff or soft brushes. A wool buffing wheel is used to achieve a high lustre.

**Cleaning**

Following polishing the surface of the restoration is cleaned with steam or ultrasound and completely washed with a soapy solution.

**Warning**

- > Must be kept out of the reach of children
- > Irritating to eyes and skin
- > Prevent contact with eyes and skin
- > Do not inhale vapours.
- > In case of accidental contact with eyes, wash immediately with plenty of water and seek medical attention.
- > In case of accidental contact with skin, immediately wash with plenty of water.
- > If swallowed do not induce vomiting. Immediately seek medical attention and show label or packaging.
- > Prevent contamination of sewerage systems and water

**Storage**

- > Storage temperature: 12–28 °C / 54–82 °F

This material has been developed solely for use in dentistry. Processing should be done strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions.

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**CANDULOR AG**