# STATIK POINTER

INSTRUCTIONS FOR USE

---

GEBRAUCHSINFORMATION

ANWENDUNG / APPLICATION / UTILISATION / USO / GEBRUIK / ПРИМЕНЕНИЕ

1.0

2.0

3.0

4.0

5.0

6.0

# STATIK POINTER

- DE GEBRAUCHSINFORMATION
- EN INSTRUCTION FOR USE
- FR MODE D'EMPLOI
- IT ISTRUZIONI D'USO
- ES INSTRUCCIONES D'USO
- NL GEBRUIKSAANWIJZING
- RU ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ

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We are delighted that you have decided to purchase a branded product manufactured by our company, CANDULOR.

This device is a technical aid which supports the dental technician when checking patient-specific analysis points. This device is distinguished by quality and a high level of precision. However, if used incorrectly, this can lead to risks. Please observe the instructions for use.

### 1. Guarantee

We grant you 2-year guarantee on material breakage of original parts from the date of purchase. Proper use according to the instructions for use is assumed.

### 2. Safety cautions and danger warnings

The device has been developed for dental use. Initial operation and handling must be conducted according to the instructions for use. The manufacturer does not accept liability for damages caused by use other than directed or improper handling. The user is also responsible for confirming that the device is suitable for the intended purpose before use, particularly if the purpose is not listed in the instructions for use.

*Please also read the safety cautions for the included laser pointer carefully.*

### 3. Product description

The Static Pointer is an instrument to indicate the autonomous chewing stability of the positioned posterior teeth. This allows preparing the positional stability of the entire denture. The Static Pointer offers the dental technician a reproducible method to ensure the position of the posterior teeth on the bone foundation.

Added value is provided by the extremely simple checking of the position of fabricated dentures and the transferring of patient-specific parameters from the model to the dental restoration.

### 3.1 Indication

- Checking autonomous chewing stability
- Using the measuring scale on the base plate, the vestibular extension of the maxillary anterior teeth in relation to the papilla incisiva is easy to check
- Checking the incisor position in relation to the first palatal fold
- Adopting markings from the bite template, for example, patient center and position of canine to the master model
- Checking the terminal tooth in relation to the determined positioning stop
- Planning of implant positions in relation to esthetic try-in

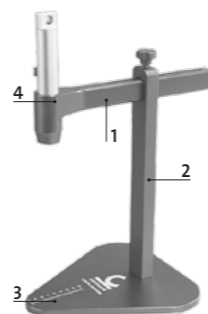
### 3.2 Contraindication

- None

### 4. Installation and initial operation

Remove the parts of the device from the box and check the scope of delivery. If parts are missing or are damaged, please contact customer service. We recommend keeping the packaging for possible shipping purposes.

Place the red horizontal arm (1) in the provided longitudinal hole of the vertical arm (2). The round opening of the horizontal arm points upwards directly over the millimeter scale of the base plate (3). Place the batteries in the laser pointer and place it in the provided opening (4) of the red horizontal arm.

### 5. Application (see cover)

#### > Picture 1.0

To ensure perfect measurement on the base plate of the measurement scale, the laser point needs to be aligned first with the scale by rotating the laser pointer in the holder.

#### > Picture 2.0

By loosening the screw in the vertical arm, the horizontal arm can be moved forwards and backwards in the longitudinal hole.

#### > Picture 3.0

Position the base of the articulator or the model below the Static Pointer.

#### > Picture 4.0 / Picture 5.0

Autonomous chewing stability: The laser beam is aligned to the cusps or the fossa. Then the dentures are removed and it is checked whether the laser point strikes the bone base.

#### > Picture 6.0

A millimeter scale on the base plate can, for example, be used to determine the overbite of the anterior teeth.

### 6. Scope of delivery

Description	Art. no
Static Pointer	662511

1 × Base plate with vertical arm incl. locking screw  
 1 × Horizontal arm  
 1 × Laser Pointer incl. 3 × LR44 coin cell batteries and safety cautions  
 1 × Instructions for use

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**CANDULOR AG**