CRS SET 15

REGISTRIERSET/REGISTRATION SET

INSTRUCTIONS FOR USE

GEBRAUCHSINFORMATION

INHALTSVERZEICHNIS / CONTENTS

		SEITE / PAGE	
DE	Gebrauchsinformation	4	<u>DE</u>
EN	Instructions for use	26	<u>EN</u>
FR	Mode d'emploi	48	<u>FR</u>
IT	Istruzioni d'uso	70	<u>IT</u>
ES	Instrucciones de uso	92	<u>ES</u>
NL	Gebruiksaanwijzing	114	<u>NL</u>
RU	Инструкция по применению	136	<u>RU</u>

CRS SET 15 REGISTRIERSET

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CRS SET 15

REGISTRATION SET

Dear customer,
We are delighted that you have decided to purchase the CANDULOR CRS Set 15 bite registration set. This registration set is a technically advanced high quality product manufactured according to state-of-the-art technology. However, if used incorrectly, this can lead to risks. Please observe the notes and read the instructions for use.

We hope you enjoy working with the product and wish you every success.

Notes on the instructions for use

Applicable registration: CRS Set 15

Target group:
dentists, dental professionals, dental technicians

These instructions for use serve the safe, appropriate and economical use of the registration.

In case of loss, the instructions for use can be obtained from the service point or downloaded as PDF from the Internet at www.candulor.com.

Safety notes

We recommend this section to all persons working with the registration or performing maintenance and cleaning work on the registration.

Use as intended

When manipulating with the CRS Set 15 on the patient, always ensure that the patient's tongue is not injured by the writing tip of the pen.

Risks and hazards

Aspiration and suffocation hazards

As the registration includes small parts which can become loose if not fixated properly, there is a risk that these parts can be swallowed.

Injury hazard

To avoid injury due to the pen, the patient must be instructed to place his/her tongue under the pen plate.

Safety cautions and danger warnings

The CRS Set 15 may only be used exclusively for the scope described in the following section.

The intended use also includes:

- > Observing the instructions, regulations and notes in these instructions for use.
- > Correct maintenance and cleaning of the registrations.

Description of design and functions

The CRS Set 15 is an intraoral support pin registration for the individual determination of the centric jaw position in edentulous and partly dentulous patients.

The basic principle of the intraoral support pin method is based on a concept by McGrane, Gysi, Gerber, etc. of a three-point support of the two jaw joints and the registration pin, which is placed in the center of the mandible. The support pin draws the typical image of an arrow point on a colored registration plate when the patient performs protrusion or limit movements laterally to the left and right from a dorsal position. The desired central position can then be fixated

with the fixation plate and then the entire registration can be coded three-dimensionally with a suitable material (e.g. registration silicone or plaster). Then the registration can be forwarded to the laboratory for further processing.

Indication

The registration set is designed for checking full dentures and the registration of the joint-related central position of dentulous and partially dentulous jaws.

Contraindication

The registration cannot be used for edentulous situations.


Scope of delivery

Please check whether the scope of delivery is complete.

662521 – CRS Set 15

- > 662522 – Registration plate 1 1 piece
- > 662523 – Registration plate 2 1 piece
- > 662524 – Registration plate 3 1 piece
- > 662525 – Registration plate 4 1 piece
- > 662526 – Supporting pin plate 1 piece
- > 662527 – Registration tip with sleeve short 1 piece
- > 662528 – Registration tip with sleeve long 1 piece
- > 662529 – Plexiglas roundel 1 piece
- > 696276 – CANDULOR Marker 1 piece
- > Instructions for use 1 piece

CRS SET 15 – COMPONENTS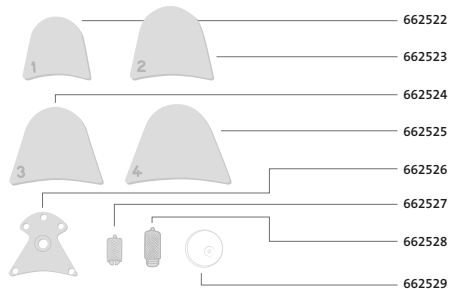

Please clean the individual components before initial use.

Processing the CRS Set 15 registration Mandibular template (Fig. 1)

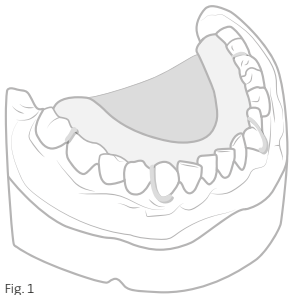
Fig. 1

1. Mark the expansion of the mandibular carrier base and bracket patterns.
2. Block out undercut regions on the model (teeth and mucosa).
3. It is recommended to fixate the registration plate passively in the mouth with brackets.
4. Align registration plate to the occlusal plane using soft wax.
5. Carefully remove the coded registration plate with the wax.
6. Isolate model with ISO-K.
7. Adapt acrylic base with CANDULOR Ultra Tray or C-Plast and surround bent brackets with sufficient acrylic.
8. Reposition the registration plate over the wax key and then frame into base.

Processing the CRS Set 15 registration

Maxillary template (Fig. 2)

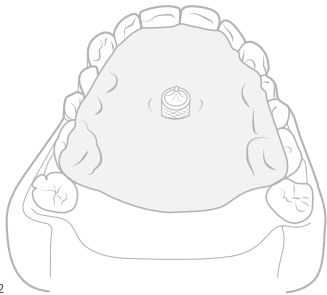
Fig. 2

1. Steps 1 – 3 analog to the mandible.
2. Adapt the base with CANDULOR Ultra Tray or C-Plast tray material according to the marked expansion.
3. Integrate the maxillary supporting pin plate into the base such, that the registration tip is in regio 15/25 and parallel to the occlusal plane. Alternatively, the registration tip can be incorporated directly into the acrylic base with the sleeve.

Attention: the registration tip must be accessible basally to turn the screw. If necessary, block out with wax.

In the upper palatal plate, the palatal cusps must be mounted so that the tooth rows cannot be displaced under pressure. The buccal cusps must remain free for later coding in the mouth. The palatal plate must be recessed basally at the threaded sleeve so that the height of the registration tip can be adjusted.

4. The mandible and maxilla templates prepared as described in the processing instructions are checked to ensure they fit properly. The patient must sit upright and keep his/her straight for bite registration.

5. The support pin of the maxillary plate is adjusted until there is no longer any contact between the upper and lower rows of teeth during the movements.

6. While sitting in the upright position, the patient is instructed to move his mandible back and forth as far as possible while touching pin and registration plate. This procedure is repeated 2 – 3 times.

7. The patient is then asked to move his mandible to the left and then back to the center again. The patient performs the same movement on the right side two to three times. This exercise is to prepare the patient for the actual bite registration procedure.

8. Using the CANDULOR Marker provided, generously paint the mandibular registration plate.

9. The templates are then put in and, following the dentist's instructions, the patient traces the angle by anterior – posterior and lateral movement of the mandible.

10. Crosshair symbol drawn on the tip of the arrow (centric).

11. To check the gothic arch, the entire plate except for the crosshairs is painted again and the entire bite registration procedure is repeated.

12. If the second gothic arch is also positioned exactly on the crosshairs, the Plexiglas roundel can be drilled in the position of the tip of the arrow and fixed into place using adhesive wax.

13. The opening and closing movements make it possible to check in the mouth and see whether the tip of the arrow snaps into the hole of the Plexiglas roundel (Fig. 3).

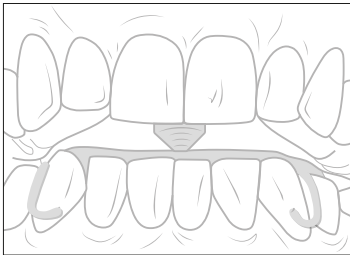
Fig. 3

14. If this is the case, the position is cemented using registration silicone or impression plaster.

15. If this is not the case, the registration procedure must be repeated.

Notes on cleaning

What	When	With what
Registration parts and screws (metal)	After each use	Sterilization in the autoclave at 134° C for 5:30 minutes, perform pre-vacuum method. Please observe the applicable standards.
Plexiglas roundels (acrylic)	After each use	Perform disinfection in the immersion bath according to the instructions of the disinfectant manufacturer.

Notes on care

Avoid any contact with strong acids and solvents (e.g. MMA, acetone). This prevents any damage to the surface treatment.

Clean, sterilize or disinfect the parts after every use or contact with saliva or blood.

- > Only use disinfectants suitable for stainless steel and light metals.
- > Avoid any contact with strong acids, alkalis or solvents.
- > Clean with ultrasonic bath, water, steam jet or sterilization equipment.

Warning

The registration has been developed for dental use. Initial operation and handling must be conducted according to the instructions for use. CANDULOR does not accept liability for damages caused by use other than directed or improper handling. The user is also responsible for confirming that the registration is suitable for the intended purpose before use, particularly if the purpose is not listed in the instructions for use.

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CANDULOR AG