CRS SET 20

REGISTRIERSET/REGISTRATION SET

INSTRUCTIONS FOR USE

GEBRAUCHSINFORMATION

INHALTSVERZEICHNIS / CONTENTS

		SEITE / PAGE	
DE	Gebrauchsinformation	4	<u>DE</u>
EN	Instructions for use	26	<u>EN</u>
FR	Mode d'emploi	48	<u>FR</u>
IT	Istruzioni d'uso	70	<u>IT</u>
ES	Instrucciones de uso	92	<u>ES</u>
NL	Gebruiksaanwijzing	114	<u>NL</u>
RU	Инструкция по применению	136	<u>RU</u>

CRS SET 20 REGISTRIERSET

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CRS SET 20 REGISTRATION SET

Dear customer,
We are delighted that you have decided to purchase the CANDULOR CRS Set 20 bite registration set. This registration set is a technically advanced high quality product manufactured according to state-of-the-art technology. However, if used incorrectly, this can lead to risks. Please observe the notes and read the instructions for use.

We hope you enjoy working with the product and wish you every success.

Notes on the instructions for use

Applicable registration: CRS Set 20

Target group: dentists, dental professionals, dental technicians

These instructions for use serve the safe, appropriate and economical use of the registration.

In case of loss, the instructions for use can be obtained from the service point or downloaded as PDF from the Internet at www.candulor.com.

Safety notes

We recommend this section to all persons working with the registration or performing maintenance and cleaning work on the registration.

Use as intended

When manipulating with the CRS Set 20 on the patient, always ensure that the patient's tongue is not injured by the writing tip of the pen.

Risks and hazards

Aspiration and suffocation hazards

As the registration includes small parts which can become loose if not fixated properly, there is a risk that these parts can be swallowed.

Safety cautions and danger warnings

The CRS Set 20 may only be used exclusively for the scope described in the following section.

The intended use also includes:

- > Observing the instructions, regulations and notes in these instructions for use
- > Correct maintenance and cleaning of the registration

Description of design and functions

Compared with classical support pin registry, closing movements on a frontal plateau permit virtually relaxed positioning of the tongue in the base of the mouth. Current research results have demonstrated that, when applying techniques with slow and ballistic closing movements

on a frontal plateau, the position of the jaw more or less precisely follows the movement path of a physiological closing movement. Thus, the therapeutic final bite position achieved with this technique follows the path of a physiological closing movement, allowing the patient interference-free acceptance immediately after placement of the prosthesis or the bite block. This reduces the unavoidable adaptation of the neuro-muscular system to a minimum. For this reason, these techniques are also suitable for determining a therapeutic jaw relationship during functional treatment. The techniques can also be employed for eden-

tulous patients as an alternative to the classical support pin registry. The process of reocclusion in the articulator, with the inherent procedural error of pure rotation around a stationary hinge axis, can thus be avoided as the dentist performs the grinding measures directly in the patient's mouth.

Indication

The registration set is designed for checking full dentures and the easy determination of the jaw relationship of dentulous and partially dentulous jaws.


Scope of delivery

Please check whether the scope of delivery is complete.

681829 – CRS Set 20

- > 681830 – Registration plates small 2 pieces
- > 681831 – Registration plates large 2 pieces
- > 662527 – Registration tip with sleeve short 1 piece
- > 662528 – Registration tip with sleeve long 1 piece
- > 696276 – CANDULOR Marker 1 piece
- > Instructions for use 1 piece

CRS SET 20 – COMPONENTS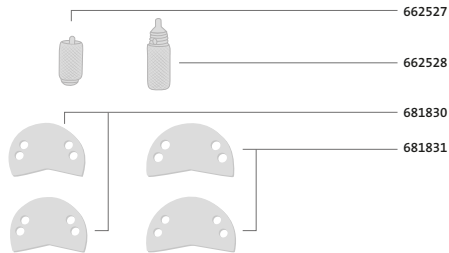

Please clean the individual components before initial use.

Preparing the CRS Set 20 registration (in the laboratory)
Mandibular template (Fig. 1)

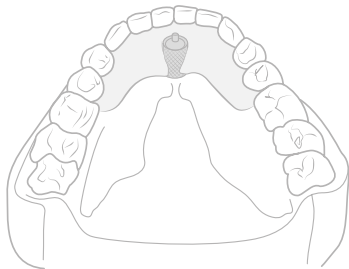
Fig. 1

1. Block out undercut regions on the model (teeth and mucosa) with wax.
2. Isolate model with ISO-K.
3. Prepare an acrylic base lingually in the region of the mandibular anterior teeth with CANDULOR Ultra Tray or C-Plast, and then mount the support pin immediately lingual to the lower central incisors vertical to the occlusal plane.

Preparing the CRS Set 20 registration (in the laboratory)
Maxillary template (Fig. 2)

Fig. 2

1. Block out undercut regions on the model (teeth and mucosa) with wax.
2. Isolate model with ISO-K.
3. Prepare an acrylic base palatal in the region of the maxillary anterior teeth with CANDULOR Ultra Tray or C-Plast, and then mount the registration plate parallel to the occlusal plane.

Using the CRS Set 20 (on the patient)

The patient must sit upright and keep his/her straight for bite registration.

1. The mandible and maxilla templates are checked to ensure they fit properly. They must be mounted stable and tilt-free. If necessary, this can be accomplished by using cyanacrylate adhesive (superglue) or temporary cement. In case of full dentures, the base of the mandibular prosthesis should be secured in its position with dental adhesive to securely stabilize the prosthesis during wide opening movements.

2. The vertical jaw relationship is prepared according to therapeutic requirements via the registration tip in the mandible.

3. The patient performs approx. 10 – 15 opening and closing movements on the plateau. Ideally speaking, the dentist's instructions should be: »Please open and close the lower jaw in a loose and relaxed manner«. In this context, it is adequate for the treating professional to recommend a tact frequency of approx. 1 Hz. Opening should be approx. 70 – 80% of maximum movement capacity.

4. Following training, an indicator wax (e.g. Occlusal Indicator Wax by Kerr) can be applied to the plate in the maxilla. The patient performs a further 3 – 5 closing movements following the same instructions, whereby the pin should only leave a single impression in the wax, or the plate can be marked using the CANDULOR Marker. (Fig. 3 and 4).

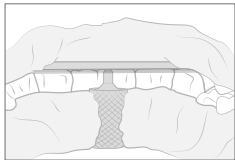
Fig. 3

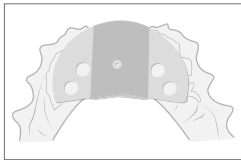
Fig. 4

5. This can now be followed by applying a malleable registry material to the row of teeth and relaxed closing of the jaw again. After this registration procedure, the wax on the maxillary template can be checked to see whether the patient contacted the existing impression again or whether he made a new impression in the wax due to an irritation. In the latter case the registration procedure should be repeated.

The silicone bite registrations must be reduced in the laboratory until only the impressions of the cusp tips remain in the registration.

6. The uncoordinated patient

If a patient is not able to perform reproducible closing movements, i.e. due to pain, one can switch to the technique of fast, ballistic closing movement. Here the jaw is also opened to approx. 80 % of the maximum movement capacity and initially closed slowly and then quickly for the last 10 mm (Fig. 5). After a few repeat trainings of this procedure, the registration material is then applied directly, followed by the quick closing movement. This registration serves as temporary therapeutic position. Following completion of functional pretreatment, a final therapeutic jaw relationship should be performed with a slow closing movement.

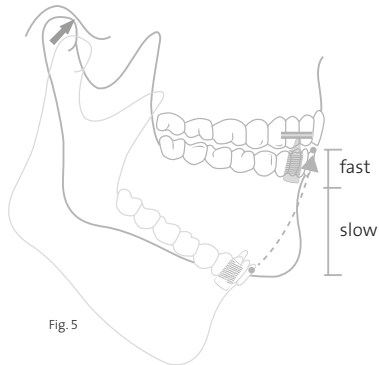
Fig. 5

7. Occlusal corrections to prosthetic restorations

Smaller occlusal corrections of dentures can be performed in the patient's mouth with the aid of CRS Set 20. This allows precise detection and removal of pre-contacts which may be irritating to the patient. To this purpose, the templates are mounted intra-orally as already described, and the support pin raised until the rows of teeth are completely disengaged. This is followed by stepwise lowering of the pin. The patient performs simple opening and closing movements (see above) and reports any occurring pre-contacts to the dentist. These contacts can be made visible using occlusion paper and then ground.

Notes on cleaning

What	When	With what
Registration parts and screws (metal)	After each use	Sterilization in the autoclave at 134° C for 5:30 minutes, perform pre-vacuum method. Please observe the applicable standards.

Notes on care

Avoid any contact with strong acids and solvents (e.g. MMA, acetone). This prevents any damage to the surface treatment.

Clean, sterilize or disinfect the parts after every use or contact with saliva or blood.

- > Only use disinfectants suitable for stainless steel and light metals.
- > Avoid any contact with strong acids, alkalis or solvents.
- > Clean with ultrasonic bath, water, steam jet or sterilization equipment.

Warning

The registration has been developed for dental use. Initial operation and handling must be conducted according to the instructions for use. CANDULOR does not accept liability for damages caused by use other than directed or improper handling. The user is also responsible for confirming that the registration is suitable for the intended purpose before use, particularly if the purpose is not listed in the instructions for use.

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CANDULOR AG