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Safety instructions

Seite/Page

DE	Sicherheitshinweise für CANDULOR Prothesenzähne	1
EN	Safety instructions for CANDULOR denture teeth	2
FR	Consignes de sécurité relatives aux dents prothétiques CANDULOR	3
IT	Avvertenze di sicurezza per i denti protesici CANDULOR	4
ES	Indicaciones de seguridad para los dientes artificiales CANDULOR	5
NL	Veiligheidsinstructies voor prothese-elementen van CANDULOR	6
RU	Инструкции по технике безопасности при работе с искусственными зубами CANDULOR	7

DE

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EN

Safety instructions for CANDULOR denture teeth

PMMA (TCR, Preference), Composite (NFC+) and Ceramic (CT Porcelain)

Minimum layer thickness

It must be ensured that the stability of the teeth and the correct shade effect are maintained. It is therefore recommended to observe the minimum layer thickness.

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To ensure that removable dentures remain functional over longer periods of time, the following prerequisites should be met:

- The dentures should be planned and fabricated in close cooperation with the dentist.
- Dentist and dental technician should ensure a balanced occlusion and articulation.
- All dental products should be used according to the instructions of the manufacturer. Denture teeth, should not be weakened by excess grinding.
- In order to ensure a reliable bond for PMMA/ Composite teeth with the denture base material, the teeth are cleaned with steam, roughened and wetted with monomer. When completing the denture with an auto-curing polymer, we also recommend using mechanical retentions.
- The application of a glaze varnish or denture cleaning spray is not recommended.
- The teeth should not come into contact with a flame or direct heat (for example alcohol torch).
- Heat development as a result of too much pressure during grinding must be prevented.
- Cleaning is performed with commercial cleaning agents. Contact with solvents must be prevented.
- If an allergy against components of the product is known, prefabricated teeth should not be used.
- When using C&B veneering materials, the corresponding Instructions for Use must be observed.
- We recommend wearing a face mask and using extraction equipment for protection against grinding dust.
- We recommend wearing protective goggles to protect your eyes from grinding chips.

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